

Quick Tip Electrode Prep + Placement


ERG Preparation:

CLEAN

Cleanse areas where electrodes will be placed with OCUSOFT[®].

PREP

Exfoliate with NuPrep[®] on areas just cleansed, taking care along the lashline, starting at the nose and wiping towards the temple.

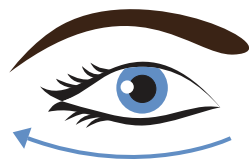


WIPE + DRY

Remove remaining NuPrep with tissue or gauze.

APPLY

Adhere electrodes to prepped areas.

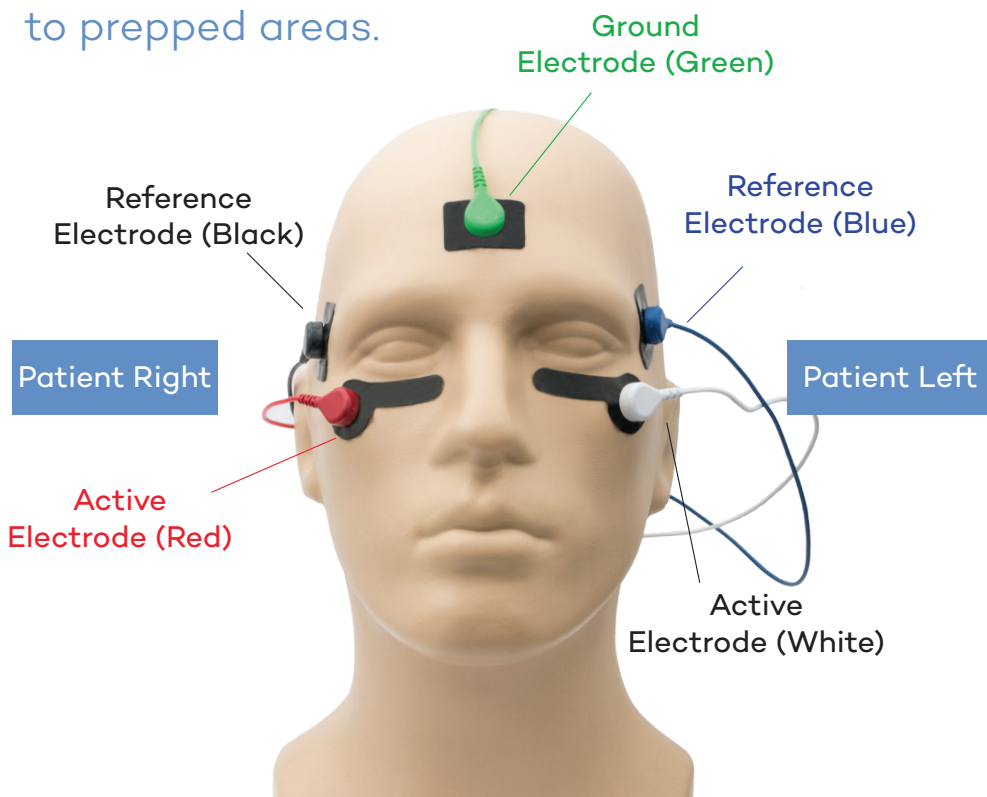

VEP Preparation:

CLEAN

Cleanse forehead with OCUSOFT.

PREP

Part hair down the middle and place the T-Band around the patient's head. Exfoliate with NuPrep on areas where electrodes will be placed, wiping in one direction 5x on the forehead and on both sides of parted hair.



WIPE + DRY

Remove remaining NuPrep with tissue or gauze.

APPLY

Snap cup and pellet into red and black leads, then snap electrodes into keepers.

Fill scalp electrodes with ELEFIX[®] and adhere red and black electrodes to previously prepped area on scalp.



Adhere green electrode to previously prepped forehead area.

VEP + ERG Supplies:

